


arkray

Crystal[®] HIV

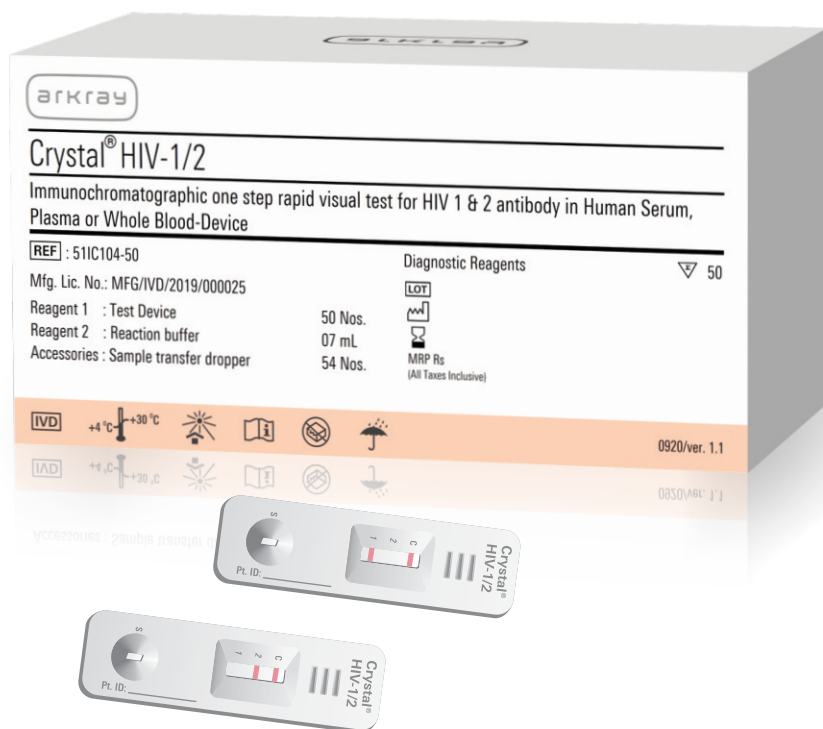

Rapid visual

immunochromatographic test for detection of

HIV-I (including group-O) and **HIV-2** antibodies


Crystal[®] HIV

Lateral flow immunochromatographic rapid visual test for HIV-1 and HIV-2 antibodies in human serum, plasma or whole blood-device.

■ *Clear & Differential Detection of HIV-1 & HIV-2*



Recombinant gp41, gp120 & gp36 antigen, detects low titre antibodies in early sero-conversion stage.

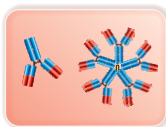


Can react with group-O and subtypes A, B, C, D, G, H, I, J, AG & CPX.

■ *Reliable Results & Convenient Storage*




Sensitivity	Specificity
100%	100%

Data on Arkray files

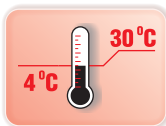


Can be stored at room temperature (4°C to 30°C), suitable for field use.

■ *Simple Test Procedure*

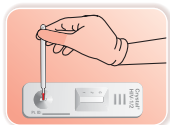

Add 1 drop (10µL) serum / plasma or 2 drops (20µL) capillary / venous blood in sample window "S".

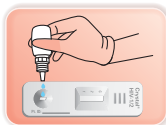

Add 2 drops reaction buffer in sample window "S" and read the results within 10~20 minutes.

■ *Ordering Information*

Product	Accessory	Test	25 Test	50 Test
Crystal [®] HIV	Sample Transfer Dropper	Device	51IC104-25	51IC104-50

THE MANUFACTURING SITE'S QMS IS CERTIFIED FOR
ISO 13485:2016, ISO 9001:2015

arkray healthcare Pvt. Ltd.

Plot No. 336, 338, 340, Road No. 3, G.I.D.C., Sachin - 394 230, (Surat), INDIA.

Telephone: 0261-6167777, Facsimile: 0261-6167778, Website: www.arkray.co.in

Specifications may change without prior notice

1220 / ver. 1.0
000600000756